

PORODO
LIFESTYLE



Porodo Lifestyle
(Wearable Motorised Breast Pump)

SKU: PD-LFST016

Table of Contents

Features	2
Specifications	2
Disclaimer	3
Warnings	3
Product Overview	4
Cleaning and Disinfection	4
Operating Instructions	5
Assembly Guide	6
Various Suitable Occasions	7
Warranty	8
Contact Us	7

Before using the product, please carefully read this User Manual to guarantee correct usage and keep it secure for future reference.

Features

- 1. Hands-Free Design: The breast pump is compact and lightweight, fitting discreetly inside a bra for convenient, hands-free milk expression.
- 2. No Tubing, No Feeding Bottle: Designed without external tubes, wires, or bottles, offering a streamlined and clutter-free pumping experience.
- 3. Use Anywhere and Anytime: Portable and versatile, allowing for milk expression at any location and time without the need for prior planning.
- 4. All-In-One Device: Easy to assemble and clean, this integrated device simplifies the setup and maintenance process.
- 5. Made of Food-Grade PP and Silicone, BPA-Free: Constructed from safe, high-quality materials that are free from BPA and other harmful chemicals, ensuring safe use for both mother and baby.
- 6. Safe, Convenient, Efficient: Engineered from environmentally friendly materials, this breast pump offers a simple and efficient milk extraction method, with all detachable parts (except the motor) safe for boiling water disinfection.

Specifications

Product Name	Porodo Lifestyle Wearable Motorised Breast Pump
SKU	PD-LFST016
Input	DC 5V/1A
Battery Capacity	1200mAh
Recharge Time	2 Hours
Working Time	3 Hours
Charging Interface	Type-C
Flange Size	24mm
Collection Cup Capacity	180 ml
Expression Mode	Stimulation (5 Levels), Expression (5 Levels), Massage (5 Levels)
Product Weight	200 grams
Product Dimension	113×72×135mm
Certifications	CE, ROHS,CPC

Disclaimer


We are dedicated to enhancing the efficiency, convenience, and comfort of breastfeeding for mothers. Our innovative design ensures that even beginners can easily use the breast pump to meet the unique needs of different mothers, providing a solution that is safer, more practical, and more efficient. We strive to offer thoughtful pregnancy products that cater to every mother. We are committed to protecting and nurturing, with a focus on breast milk. We blend design aesthetics with functionality to ensure high-quality products. We hope that this breast pump enhances your breastfeeding experience.

Warnings


1. Use only the original adapter provided; do not use non-original adapters.
2. Ensure the machine is powered off after each use.
3. Do not disinfect or wash the pump motor and power adapter.
4. If you have a low supply of breast milk, use this product only under the supervision of a doctor or healthcare professional.
5. If breast milk is not expressed within 5 minutes, stop using the pump immediately.
6. Consult a doctor or breastfeeding specialist if you experience any pain or discomfort during use.
7. Do not use the breast pump during pregnancy.
8. Wash and thoroughly dry your hands before assembling the parts.
9. Avoid using harsh cleaners or abrasive products, as they may scratch or damage the item.
10. Store the product in a cool, dry place away from direct sunlight.
11. Replace any damaged parts promptly.
12. Replace the valve every two months; a spare valve is provided.
13. Do not use a cleaning brush on the valve, as it may cause damage.
14. As this is a personal care product, do not share it with others.
15. Ensure you use the correct horn cover.
16. Keep accessories out of reach of infants to prevent the risk of accidental suffocation.

Product Overview



Silicon diaphragm

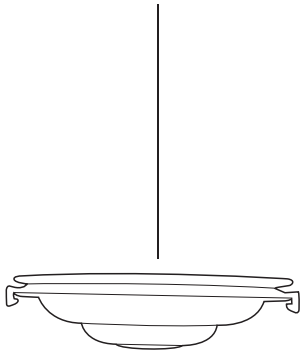


Breast shield

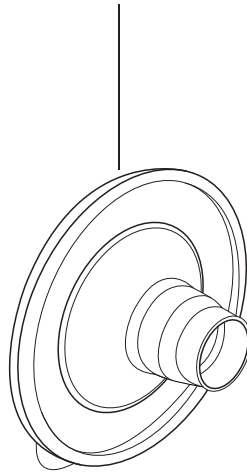


Motor pump

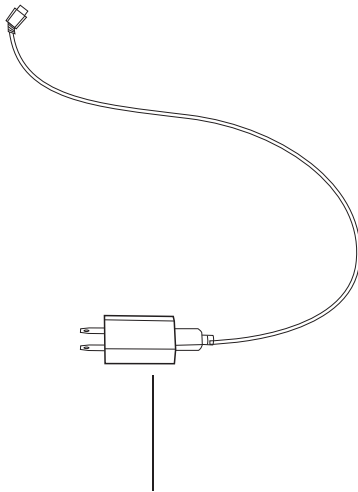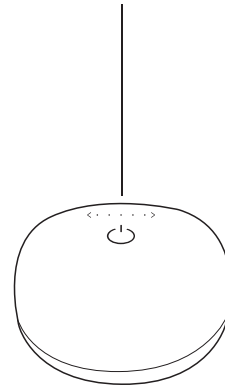


Power adapter

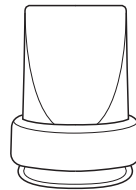

Valve

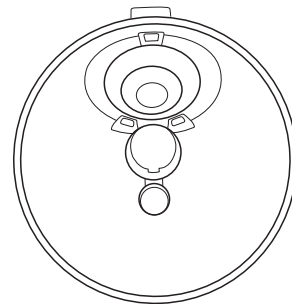

Storage tank

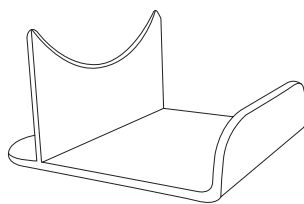

Breast pump bracket

Cleaning and Disinfection

[A] Before First Use

1. Disassemble all accessories.
2. Wash them with warm water.
3. Rinse thoroughly.
4. Sterilize the accessories by boiling them for 5-3 minutes.
5. Use a large pan to prevent the accessories from touching the bottom or sides.

6. Place the accessories on a clean cloth or a drying rack to air dry.

Note: The motor pump is an electrical component and cannot be washed or disinfected.

[B] After Use

- 1.** Disassemble all accessories (excluding the motor pump).
- 2.** Rinse the accessories with cool water immediately after use.
- 3.** Wash the accessories with warm water.
- 4.** Gently clean the silicone diaphragm with your fingers.
- 5.** Rinse thoroughly.
- 6.** Place the accessories on a clean cloth or a drying rack to air dry.

[C] Method of Disinfection

- 1.** Follow the cleaning steps outlined from steps [B].1 to [B].5.
- 2.** Sterilize the accessories by boiling them for 5-3 minutes.
- 3.** Use a large pan to prevent the accessories from touching the bottom and sides of the pan.
- 4.** After disinfection, place the accessories on a clean cloth or a drying rack to air dry.

Operating Instructions

[1] Buttons and Display

- a.** Suction Level/Battery Level Indicator: Shows the current suction strength and battery status.
- b.** Decrease Suction Power Button: Press to reduce the suction power.
- c.** Increase Suction Power Button: Press to increase the suction power.
- d.** Power Button: Press to turn the device on or off.

e. Suction Mode Indicator

Orange Light: Indicates Stimulation Mode.


Green Light: Indicates Expression Mode.

Chartreuse Light: Indicates Mixed Mode.

f. Charging Indicator

Flashing Green Light: The device is charging.

Solid Green Light: The device is fully charged.

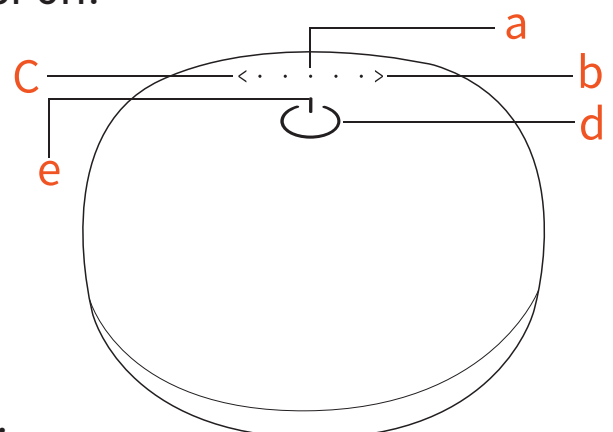


[2] Operation Method


1. To Power On: Long press the power button. The display will initially show the battery level, then default to the last used suction mode and level.
2. To Power Off: Long press the power button.
3. To Change Suction Mode: Briefly press the power button.
4. Adjusting Suction Level: Each suction mode offers five levels of strength.
 - 4.1 Stimulation Mode: Press the "<" or ">" buttons to increase or decrease suction strength.
 - 4.2 Expression Mode: Press the "<" or ">" buttons to adjust suction strength.
 - 4.3 Mixed Mode: Press the "<" or ">" buttons to modify suction strength and frequency.
5. Checking the Battery Level
 - Method 1: Simultaneously hold down both the "<" and ">" buttons. The battery level indicator will flash three times.
 - Method 2: The battery level indicator will automatically flash three times when you turn on the breast pump.

Tips: If breast milk is not extracted quickly, do not worry and try to relax. If the issue persists for more than 5 minutes, stop and try again later.


Assembly Guide

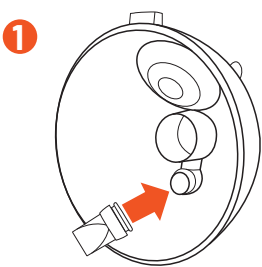


Position the valve into the designated slot on the pump body.

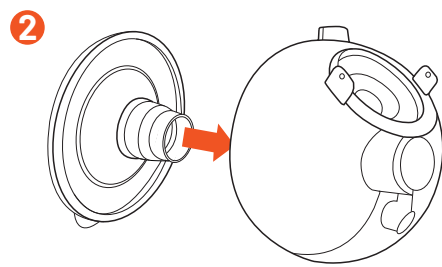

Attach the silicone horn to the valve already placed in the pump.

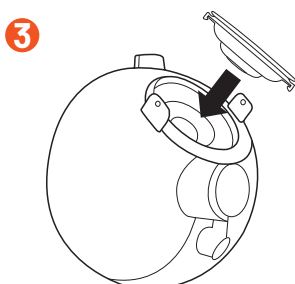


Place the silicone diaphragm over the assembly.

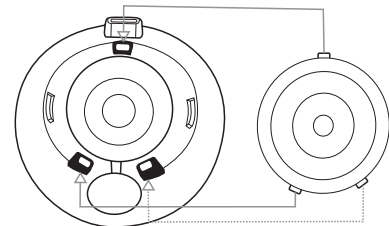


Secure the buckle of the silicone diaphragm to the connector to ensure a proper seal.


4

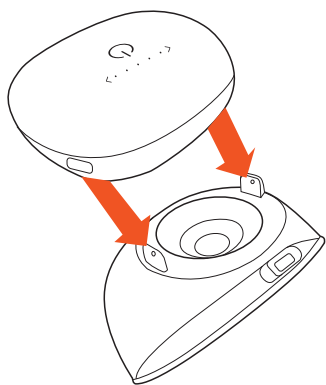


Attach the motor pump unit to the base of the assembled parts, ensuring it is securely fastened.

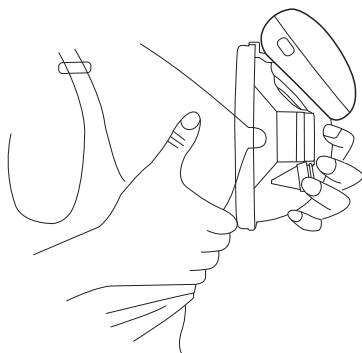

5


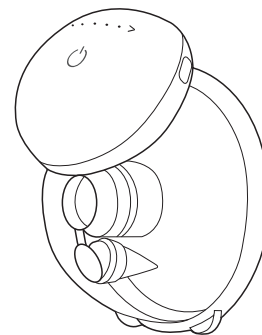


Verify that all components are correctly assembled and secure. The breast pump should now be fully assembled.

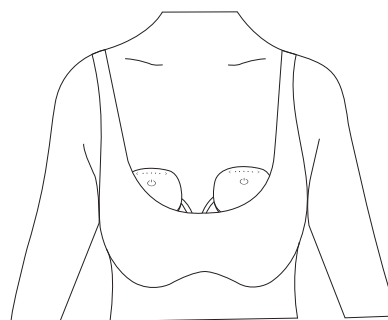

Place the assembled breast pump in the appropriate position on the bra for use.

Various Suitable Occasions

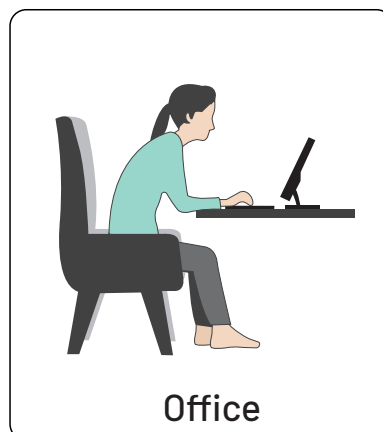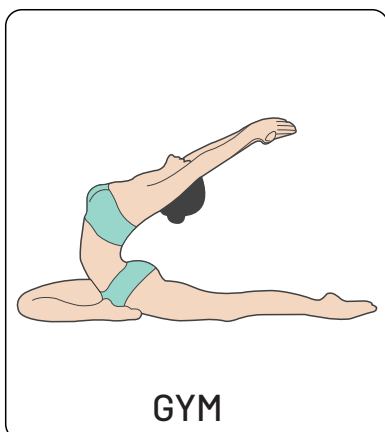

Warranty

Products that you buy directly from our **Porodo** website or shop come with a 24-month warranty.

When you buy **Porodo** products from any of our approved sellers, you only get a 12-month warranty. If you want to extend this warranty, go to our website at **porodo.net/warranty** and fill out the form with your information. Don't forget to upload a picture of the product too. After we've checked and accepted your request, we'll send you an email to confirm that your product's warranty has been extended.

For more info, please check:
porodo.net/warranty

Contact Us

If you have any questions about this Privacy Policy, please contact us at:
info@porodo.net

Website: **porodo.net**

Service Support: **support@porodo.net**

Instagram: **porodo**